melcareTM

+++++

MANUKA⁺ WOUND GEL[®]

+++++

Instructions For Use

Mode d'emploi

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Modo de uso

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Användning

Brugsanvisning

Instruktioner

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+++++

REF: M0122

 **Melcare Biomedical Pty Ltd**
451 West Mt Cotton Rd
MOUNT COTTON QLD 4165
AUSTRALIA

UK REP

Melcare (Europe) Ltd
Carpenter Court
1 Maple Rd, Bramhall
STOCKPORT SK7 2DH
ENGLAND

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IFU0122
V04092025

Instructions For Use

Melcare's Manuka+ Wound Gel™ is a sterile dressing made from *Leptospermum* sp. honey (80%), plant derived hydrocolloids and a natural oil.

PRODUCT PROPERTIES

- Provides an antibacterial wound environment;
- Helps maintain wound pH;
- Assists in removal of slough and necrotic tissue;
- Rapidly removes malodour;
- Is non-adherent thus reducing trauma at dressing change.

INDICATIONS

Manuka Wound Gel™ is indicated as a contact layer for wounds such as:

- Venous ulcers;
- Pressure ulcers (I-IV);
- Diabetic Ulcers;
- 1st and 2nd degree burns;
- Surgical wounds
- Donor and recipient graft sites
- Sloughy, malodorous wounds
- General First Aid

DIRECTIONS FOR USE

1. Inspect and clean the wound using standard techniques.
2. Twist off cap and remove security seal.
3. Manuka Wound Gel™ should be applied liberally (e.g. for a 10cm x 10cm wound apply 20-25g of Manuka Wound Gel).
4. Manuka Wound Gel™ can be applied either directly to the wound or to a suitable dressing.
5. Avoid direct contact with the nozzle of the tube.
6. A non-adherent gauze should be applied to cover the Manuka Wound Gel™ with additional secondary dressings applied as required for absorbance of wound exudate.
7. Wound margins can be protected by a suitable barrier cream (Melcare's Derma Care™)
8. The frequency of changing will depend on the clinical circumstances and this can vary from more than once per day for draining wounds to up to 7 days for unsoiled dressings.

9. Manuka Wound Gel™ can be removed from the wound by rinsing with clean saline or tap water at dressing change.

OBSERVATIONS

Manuka Wound Gel™ provides natural debridement of the wound. The wound may therefore appear larger after the initial dressing change. The honey present in Manuka Wound Gel™ will be gradually diluted by exudate and absorbed into the secondary dressing.

CONTRAINDICATIONS/ PRECAUTIONS

Reserve tube for single patient use. Sterile unless package is opened or damaged.

Wounds subject to arterial insufficiency, diabetic conditions or signs of infection should be managed by a healthcare professional.

Any experience of a reaction other than a short lasting stinging sensation, remove the dressing and wash the affected area. Not for use on heavily exuding wounds.

Manuka Wound Gel™ is not recommended for body cavities. DO NOT use on patients with known hypersensitivity to honey. In the event of a slow or non-responding wound please seek medical advice.

Discard 3 months after opening

REPORTING

Any serious incident (death of a user or serious deterioration of a patient's health) in relation to this device should be reported to the manufacturer at the front of this document and the competent authority of the Member State in which the user and/or patient is established.

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